



Carmellose Sodium 0.5% Preservative free

What is PF Drops Carmellose Sodium 0.5%?

This product contains carmellose sodium, a viscous ingredient which retains moisture on the eye surface, helping to keep the eye lubricated. PF Drops Carmellose Sodium 0.5% is supplied as a preservative free, sterile eye drop solution in a 10ml multi-dose bottle. After first opening, the product should be discarded within 2 months. This product is compatible with soft contact lenses.


What is PF Drops Carmellose Sodium 0.5% used for?

PF Drops Carmellose Sodium 0.5% is used to obtain lasting and soothing relief in mild to moderate dry eyes. It also provides prolonged relief against burning, irritation and discomfort associated with ocular dryness or exposure to environmental factors.

How to use PF Drops Carmellose Sodium 0.5%?

Instill 1 drop in the affected eye(s) 2-3 times daily, as required.

Remove the tamper-evident seal on first use. Remove the bottle cap before instilling 1 drop into the eye. Put the cap back on the bottle after each use.

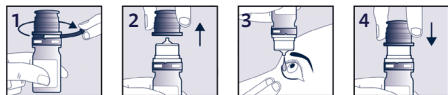

Precautions and warnings

- Do not use if the bottle or packaging is damaged.
- Do not use if you are sensitive to any of the ingredients of the product.
- Do not use after the expiry date shown on the carton and bottle.
- Do not touch the eye with the tip of the bottle.
- After administration, in rare cases, a slight temporary blurring of vision may be experienced due to the viscosity of the solution. It is therefore advisable to wait for a few seconds before undertaking any activity that requires clear vision.
- Wait at least 10-15 minutes before instilling any other ophthalmic product.
- If you wear soft contact lenses, remove them before using the product and do not replace the lenses for at least 15 minutes.
- After first opening the bottle, use the product within 2 months.
- Keep out of the sight and reach of children.
- Do not swallow.
- The product is not normally intended for use for more than 30 days; if symptoms persist consult your doctor.

Ingredients

Carmellose sodium, boric acid, sodium tetraborate, sodium chloride, potassium chloride, calcium chloride, water.



STERILE A



UK
CA 0120



Penlan Healthcare Limited,
Abbey House, Wellington Way,
Weybridge, Surrey, KT13 0TT, UK.

Distributed by: Aspire Pharma Ltd.,
Unit 4, Rotherbrook Court, Bedford Rd,
Petersfield, Hampshire, GU32 3QG, UK.

ASPIRE
P H A R M A

PF DROPS®

Carmellose Sodium 1%

Preservative free

What is PF Drops Carmellose Sodium 1%?

This product contains carmellose sodium, a viscous ingredient which retains moisture on the eye surface, helping to keep the eye lubricated. PF Drops Carmellose Sodium 1% is supplied as a preservative free, sterile eye drop solution in a 10ml multi-dose bottle. After first opening, the product should be discarded within 2 months. This product is compatible with soft contact lenses.


What is PF Drops Carmellose Sodium 1% used for?

PF Drops Carmellose Sodium 1% is used to obtain lasting and soothing relief in moderate to severe dry eyes. It also provides protection against burning, irritation and discomfort associated with ocular dryness or exposure to environmental factors.

How to use PF Drops Carmellose Sodium 1%?

Instill 1 drop in the affected eye(s) 2-3 times daily, as required.

Remove the tamper-evident seal on first use. Remove the bottle cap before instilling 1 drop into the eye. Put the cap back on the bottle after each use.

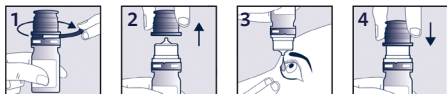

Precautions and warnings

- Do not use if the bottle or packaging is damaged.
- Do not use if you are sensitive to any of the ingredients of the product.
- Do not use after the expiry date shown on the carton and bottle.
- Do not touch the eye with the tip of the bottle.
- After administration, in rare cases, a slight temporary blurring of vision may be experienced due to the viscosity of the solution. It is therefore advisable to wait for a few seconds before undertaking any activity that requires clear vision.
- Wait at least 10-15 minutes before instilling any other ophthalmic product.
- If you wear soft contact lenses, remove them before using the product and do not replace the lenses for at least 15 minutes.
- After first opening the bottle, use the product within 2 months.
- Keep out of the sight and reach of children.
- Do not swallow.
- The product is not normally intended for use for more than 30 days; if symptoms persist consult your doctor.

Ingredients

Carmellose sodium, boric acid, sodium tetraborate, sodium chloride, potassium chloride, calcium chloride, water.



STERILE A



UK
CA 0120



Penlan Healthcare Limited,
Abbey House, Wellington Way,
Weybridge, Surrey, KT13 0TT, UK.

Distributed by: Aspire Pharma Ltd.,
Unit 4, Rotherbrook Court, Bedford Rd,
Petersfield, Hampshire, GU32 3QG, UK.

ASPIRE®
P H A R M A