

Package leaflet: Instructions for use

PF DROPS[®]

Sodium Hyaluronate 0.15%

Preservative free

What is PF Drops Sodium Hyaluronate 0.15%?

This product contains sodium hyaluronate, a lubricating and protective ingredient which improves the tear film stability, keeping the eye surface comfortable, lubricated, hydrated and protected. PF Drops Sodium Hyaluronate 0.15% is supplied as a preservative free, sterile eye drop solution in a 10ml multi-dose bottle. After first opening, the product should be discarded within 2 months. This product is compatible with soft contact lenses.


What is PF Drops Sodium Hyaluronate 0.15% used for?

PF Drops Sodium Hyaluronate 0.15% is used to obtain lasting and soothing relief from moderate to severe dry eye. It also provides prolonged relief from irritation, dry, burning and foreign body sensations in the eyes due to environmental factors such as wind, sun, dry air, salt water, smoke, excessive light, air conditioning, heating, prolonged use of a computer or in case of ocular surgery, conjunctivitis or frequent and prolonged use of contact lenses.

How to use PF Drops Sodium Hyaluronate 0.15%?

Instill 1 drop in the affected eye(s) 2-3 times daily, as required.

Remove the tamper-evident seal on first use. Remove the bottle cap before instilling 1 drop into the eye. Put the cap back on the bottle after each use.

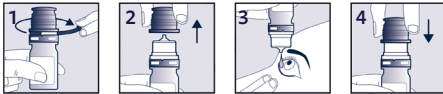

Precautions and warnings

- Do not use if the bottle or packaging is damaged.
- Do not use if you are sensitive to any of the ingredients of the product.
- Do not use after the expiry date shown on the carton and bottle.
- Do not touch the eye with the tip of the bottle.
- After administration, in rare cases, a slight temporary blurring of vision may be experienced due to the viscosity of the solution. It is therefore advisable to wait for a few seconds before undertaking any activity that requires clear vision.
- Wait at least 10-15 minutes before instilling any other ophthalmic products.
- If you wear soft contact lenses, remove them before using the product and do not replace the lenses for at least 15 minutes.
- After first opening the bottle, use the product within 2 months.
- Keep out of the sight and reach of children.
- Do not swallow.
- This product is not normally intended for use for more than 30 days; if symptoms persist consult your doctor.
- Do not use in the presence of quaternary ammonium salts.
- This product contains phosphates. In some patients with significantly damaged corneas, rare cases of calcification of the cornea during the use of eye drops containing phosphates have been reported. It is therefore not recommended to be used by patients with severe damage to the corneal surface (such as pronounced erosion or corneal ulcer).

Ingredients

Sodium hyaluronate, sodium phosphate dibasic dodecahydrate, sodium phosphate monobasic monohydrate, sodium chloride, water.



STERILE A



UK
CA 0120



Penlan Healthcare Limited,
Abbey House, Wellington Way,
Weybridge, Surrey, KT13 0TT, UK.

Distributed by: Aspire Pharma Ltd.,
Unit 4, Rotherbrook Court, Bedford Rd,
Petersfield, Hampshire, GU32 3QG, UK.

ASPIRE[®]
P H A R M A

Revision 02/2021 1010397-P.2 LFHYS10001

Package leaflet: Instructions for use

PF DROPS[®]

Sodium Hyaluronate 0.4%

Preservative free

What is PF Drops Sodium Hyaluronate 0.4%?

This product contains sodium hyaluronate, a lubricating and protective ingredient which improves the tear film stability, keeping the eye surface comfortable, lubricated, hydrated and protected. PF Drops Sodium Hyaluronate 0.4% is supplied as a preservative free, sterile eye drop solution in a 10ml multi-dose bottle. After first opening, the product should be discarded within 2 months. This product is compatible with soft contact lenses.


What is PF Drops Sodium Hyaluronate 0.4% used for?

PF Drops Sodium Hyaluronate 0.4% is used to obtain lasting and soothing relief from severe dry eye. It also provides prolonged relief from irritation, dry, burning and foreign body sensations in the eyes due to environmental factors such as wind, sun, dry air, salt water, smoke, excessive light, air conditioning, heating, prolonged use of a computer or in case of ocular surgery, conjunctivitis or frequent and prolonged use of contact lenses.

How to use PF Drops Sodium Hyaluronate 0.4%?

Instill 1 drop in the affected eye(s) 2-3 times daily, as required.

Remove the tamper-evident seal on first use. Remove the bottle cap before instilling 1 drop into the eye. Put the cap back on the bottle after each use.

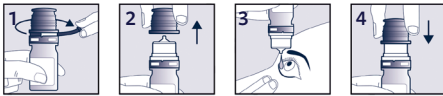

Precautions and warnings

- Do not use if the bottle or packaging is damaged.
- Do not use if you are sensitive to any of the ingredients of the product.
- Do not use after the expiry date shown on the carton and bottle.
- Do not touch the eye with the tip of the bottle.
- After administration, in rare cases, a slight temporary blurring of vision may be experienced due to the viscosity of the solution. It is therefore advisable to wait for a few seconds before undertaking any activity that requires clear vision.
- Wait at least 10-15 minutes before instilling any other ophthalmic products.
- If you wear soft contact lenses, remove them before using the product and do not replace the lenses for at least 15 minutes.
- After first opening the bottle, use the product within 2 months.
- Keep out of the sight and reach of children.
- Do not swallow.
- This product is not normally intended for use for more than 30 days; if symptoms persist consult your doctor.
- Do not use in the presence of quaternary ammonium salts.
- This product contains phosphates. In some patients with significantly damaged corneas, rare cases of calcification of the cornea during the use of eye drops containing phosphates have been reported. It is therefore not recommended to be used by patients with severe damage to the corneal surface (such as pronounced erosion or corneal ulcer).

Ingredients

Sodium hyaluronate, sodium phosphate dibasic dodecahydrate, sodium phosphate monobasic monohydrate, sodium chloride, water.



STERILE A



UK
CA 0120



Penlan Healthcare Limited,
Abbey House, Wellington Way,
Weybridge, Surrey, KT13 0TT, UK.

Distributed by: Aspire Pharma Ltd.,
Unit 4, Rotherbrook Court, Bedford Rd,
Petersfield, Hampshire, GU32 3QG, UK.

ASPIRE[®]
P H A R M A

Revision 02/2021 1010398-P9.2 LFHYS09900